

DICOM Images are located in the Student Share on Sharepoint

SanteSoft (Windows devices)

1. Open Santesoft by clicking on your “Start” button and navigating to the Sante DICOM Free Viewer
2. Open an image in Santesoft by clicking on the “File” icon, hover over “Open” and then click on “Open Series”
3. Choose a folder of interest, click on that folder; choose the study and slices from there.
4. After the slices open, close the “Open Study Series” box
5. In the viewing pane, click on or use the arrows to move to other slices.
6. Quick Demo: [Quick Start Guide](#)

OsiriX Lite (Mac)

1. Open OsiriX using “Launchpad”, or search for the program on your device
2. Open an image in Osirix by clicking on “Import a DICOM file or folder” in the upper right corner of the application.
3. Choose a folder or file of interest from the DICOM image library you downloaded. Click “Open.”
4. When the dialog box appears click “Copy files.” Be patient while the library is built in the application.
5. You can import multiple folders and files into the library, for comparison or parallel viewing.
6. Multiple tutorials: [OsiriX Viewer Support Tutorials](#)